



ViVest A1 & A3 Trainer Units

General Information

When a person has sudden cardiac arrest, the situation will be stressful, hectic and even chaotic. It is difficult, yet critical to act quickly in this environment since every second counts. For this reason, it is advised for potential users of an AED to be trained periodically with the A1 & A3 AED Trainer units which were developed especially for this purpose.

Trainer Unit Specification

Size (H x W x D) cms	23.2 x 20.9 x 5.9
Weight	1.1kg inc. battery & pads
Operation life	5 years

Trainer Unit Kit Contents

Trainer Unit	x 1
Remote Control (optional)	x 1
Electrodes Wires with Connector	x 1
Electrode Pads	x 1
Carry Case	x 1
Battery (optional)	x 4

Features and Functions

The AED-1 Trainer and AED-3 Trainer units completely simulate the operation of the real defibrillator in clinical use.

Voice Prompt and Indication Light

The trainer will guide the user during training.

The AED-1 Trainer has LED lights, a graphic panel and voice prompts.

The AED-3 Trainer unit has a colour LCD display, full colour rescue images, dual language voice prompt and text prompt.

Analysing Heart Rhythm

The trainer will analyse the heart rhythm automatically when pads are applied on to manikins properly. For paediatric manikins, apply the pads to the chest and the back as shown.

Simulation of Defibrillation

Trainer detects a shockable rhythm, then simulates to charging and instructs user to deliver a shock. Trainer detects a non shockable rhythm and instructs user to begin CPR.